# NEOTECH<sup>®</sup> Products LLC

## Neotech ChinStrap™ Respiratory Support Accessory

<b>REF</b>	Catalog No.	
	N784 Nano	N788 X-Large
	N785 Micro	N789 Macro
	N786 Small	N790 Jumbo
	N787 Large	

--

<b>EMERGO EUROPE</b>
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

<b>MD</b>	Medical Device

English	D459 Rev L 2024.12.15
Neotech ChinStrap™ Respiratory Support Accessory INDICATIONS FOR USE	
The Neotech ChinStrap is intended to help keep a patient’s mouth closed during respiratory therapy. It is intended for use on pediatric (Neonates, Infants, Children & Adolescents) and adult patients.	
DIRECTIONS FOR USE	
Step 1	

Select the proper size by measuring from the center of the chin, along the ear, across the crown of the head, along the other ear, and back to the center of the chin.

N784 Nano: fits head circumference 22-27 cm

N785 Micro: fits head circumference 28-35 cm

N786 Small: fits head circumference 33-42 cm

N787 Large: fits head circumference 36-46 cm

N788 X-Large: fits head circumference 38-53 cm

N789 Macro: fits head circumference 39-58 cm

N790 Jumbo: fits head circumference 40-66 cm

Step 2 Position the Neotech ChinStrap on the chin with the thinner portion across the front of the chin and the wider portion under the chin.

Note: Only the white side of the strap should be placed against the skin.

Step 3 Gently pull strap toward posterior crown of head with ears positioned inside ear openings and then fasten hook tab to strap. Side marked “LEFT” should be under patient’s left ear.

Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.

Step 4 Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Neotech ChinStrap should only be used while patient is under continuous direct supervision of healthcare professionals. ChinStrap can pose a potential strangulation hazard.

Discontinue immediately if skin irritation occurs.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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