


The original USA made

Little Sucker[®] and Little Sucker Cover[®]

ORAL & NASAL SUCTION DEVICE AND PROTECTIVE COVER

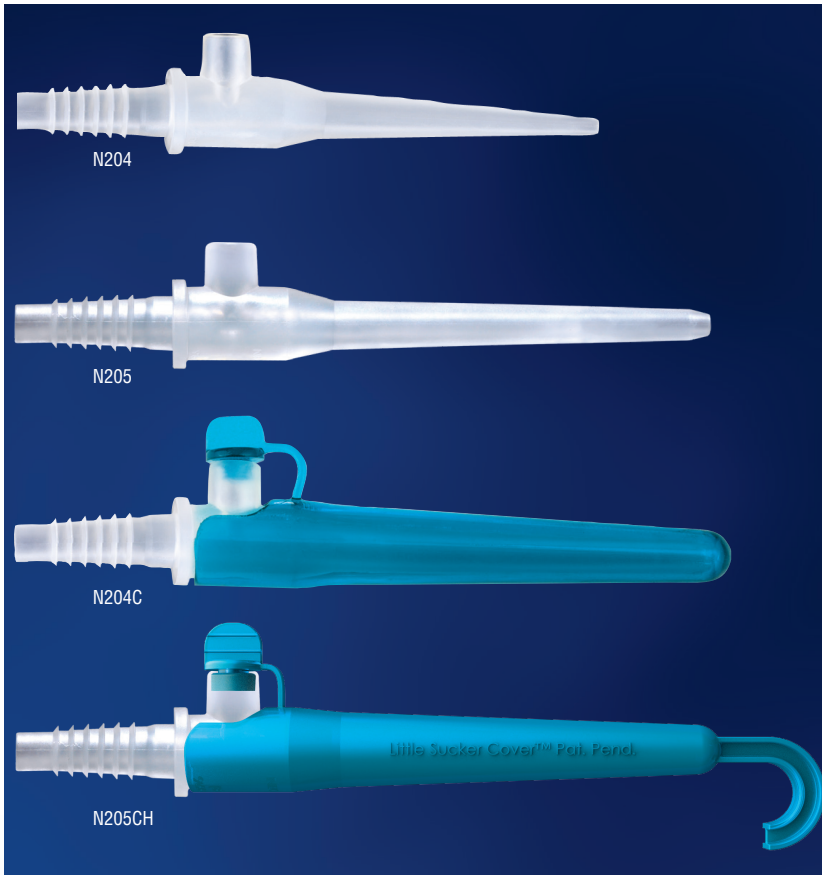

FEATURES & BENEFITS

Little Sucker

- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP

Little Sucker Cover

- Designed to protect the Little Sucker from damage, debris and external contaminants
- Convenient hook for hanging (N204CH, N205CH, N216)
- Bright blue color for better visibility
- Noise reducing port plug
- Not made with natural rubber latex or plasticizer DEHP

| CAT. # | ITEM | QTY/UNIT |
|--------|--------------------------|----------|
| N204 | Preemie | 50/box |
| N204C | Preemie with Cover | 50/box |
| N204CH | Preemie with Hook Cover | 50/box |
| N204TP | Preemie 2-piece | 50/box |
| N205 | Standard | 50/box |
| N205C | Standard with Cover | 50/box |
| N205CH | Standard with Hook Cover | 50/box |
| N205TP | Standard 2-piece | 50/box |
| N215 | Cover* | 50/box |
| N216 | Cover with Hook* | 50/box |

*Not compatible with N204TP, N205TP

U.S. Patent
 #7,635,361; #D590,056;
 #6,958,050; #D474,269;
 #D655,812



Made in USA

Little Sucker® and Little Sucker Cover®

ORAL & NASAL SUCTION DEVICE AND PROTECTIVE COVER

INDICATIONS FOR USE

The Little Sucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares.

The Little Sucker Cover is indicated for use to protect the Little Sucker.

DIRECTIONS FOR USE

- 1 If cover in place (applies to N204C/N205C), unplug thumb port and slide suction tip out of cover.
- 2 Connect barbed end of suction tip to suction line.
- 3 Set suction to appropriate pressure per hospital protocol or physician recommendation.
- 4 Block thumb port with thumb to begin suctioning.
- 5 Suction mouth or nares with intermittent suctioning. Remove thumb from thumb port to stop suctioning. Block thumb port again to continue suctioning.
- 6 Suction sterile water through suction tip after each use to clear debris.
- 7 If applicable, reinsert suction tip into cover and insert port plug of cover into thumb port of suction tip.
- 8 Replace suction tip/cover every 24 hours or per hospital protocol, whichever is sooner.

GENERAL INFORMATION

Product Materials

Little Sucker: TPE (thermoplastic elastomer)
Cover: PVC
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N204
Length: 106 mm
Tip opening diameter: 2 mm
Thumb port opening diameter: 5 mm

N204TP
Length: 116 mm
Tip opening diameter: 2 mm
Thumb port opening diameter: 6 mm

N204CH
Length: 141 mm
Tip opening diameter: 2 mm
Thumb port opening diameter: 5 mm

N205
Length: 120 mm
Tip opening diameter: 3 mm
Thumb port opening diameter: 5 mm

N205TP
Length: 130 mm
Tip opening diameter: 3 mm
Thumb port opening diameter: 6 mm

N205CH
Length: 141 mm
Tip opening diameter: 3 mm
Thumb port opening diameter: 5 mm

N215
Length: 90 mm

N216
Length: 111 mm

Packaging

Individually packaged (N204C, N204CH, N205C, N205CH: Sucker and Cover packaged together)

Package type: Polyethylene bag

Dispenser box

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For oral and nasal use only.

Inserting the Little Sucker too far into the mouth or nares can cause tissue trauma.

The Little Sucker Cover is designed for use with Little Sucker devices specifically. It may not accommodate other suction devices. It is not intended for use with the Little Sucker 2-piece devices.

The Little Sucker and Little Sucker Cover should remain free of debris.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

