

NeoSmile[™]

TEMPERATURE PROBE COVER



FEATURES & BENEFITS

- Skin friendly NeoBond Hydrocolloid base
- Designed for long lasting adhesion
- Reflective foil helps protect against ambient and radiant heat
- Cute smiley face design
- Not made with natural rubber latex or plasticizer DEHP

CAT. #	SIZE	QTY/UNIT
N731	1" Diameter	150/box



NeoSmile™

TEMPERATURE PROBE COVER

INDICATIONS FOR USE

The NeoSmile is intended to cover and secure a temperature probe.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing liner, warm NeoSmile in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.
- 2 Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.

Removal

- 1 Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.
- 2 Saturate NeoSmile with water or saline.
- 3 Slowly peel NeoSmile away from skin as you swab with water or saline.

GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid,
FLEXmark MM 200 gold foil

This product is not made with natural rubber latex or plasticizer DEHP.

Size

Diameter: 25 mm

Packaging

Standard box, no dispense feature

Perforated liner

Visit neotechproducts.com for the most up-to-date information.

Boxes are printed with environmentally friendly ink on recyclable material

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not reuse or reapply.

Single patient use only.



For more information or to order your **FREE** sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

