<b>NEOTECH</b> <sup>®</sup> Products LLC
<div> <div><span><span></span></span></div> <div><span><b>NeoSnug</b><sup>®</sup></span></div> <div><span>Tubing Holder</span></div> </div>
<b>REF</b> Catalog No.
N7014
N7015
N7025

Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

<b>EMERGO EUROPE</b>
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

<b>MD</b>	Medical Device
	
<b>CE</b>	
	
	D524 Rev D

English NeoSnug Tubing Holder
INDICATIONS FOR USE NeoSnug is intended to support, secure, and hold gastrointestinal tubes on patients' skin.
DIRECTIONS FOR USE

Preparation Clean and dry skin where NeoSnug will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSnug will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application Step 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.

Step 2 One at a time, peel NeoSnug from the clear, printed liner, place on skin, and hold in place for 60 seconds to ensure proper adhesion.

Note: If only using one NeoSnug (N7014, N7015), save second one for later use. Discard liner when done.

Step 3 Fold down the flap and peel away the white, paper liner labeled "1" from the base of the NeoSnug. This will expose the adhesive surface.

Step 4 Lay tubing across the base.

Step 5 Peel away the white, paper liner labeled "2" from the flap and fold the flap over the tubing to secure.

Note: Once closed, the NeoSnug flap is engineered to remain permanently in place. You cannot open and close repeatedly.

Removal

Step 1 Replace NeoSnug and attached tubing every 5 days or per hospital protocol, whichever is sooner.

Step 2 With the tubing still secured, saturate NeoSnug with water or saline.

Step 3 Slowly peel the NeoSnug away from the skin as you swab the skin with water or saline.

Step 4 Remove tubing per hospital protocol and discard NeoSnug and tubing.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Choking hazard. NeoSnug should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Risk of ingestion or aspiration.

Discontinue immediately if skin irritation occurs.

NeoSnug will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Disposable. Do not reuse.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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