


NeoSnug[®]

TUBING HOLDER

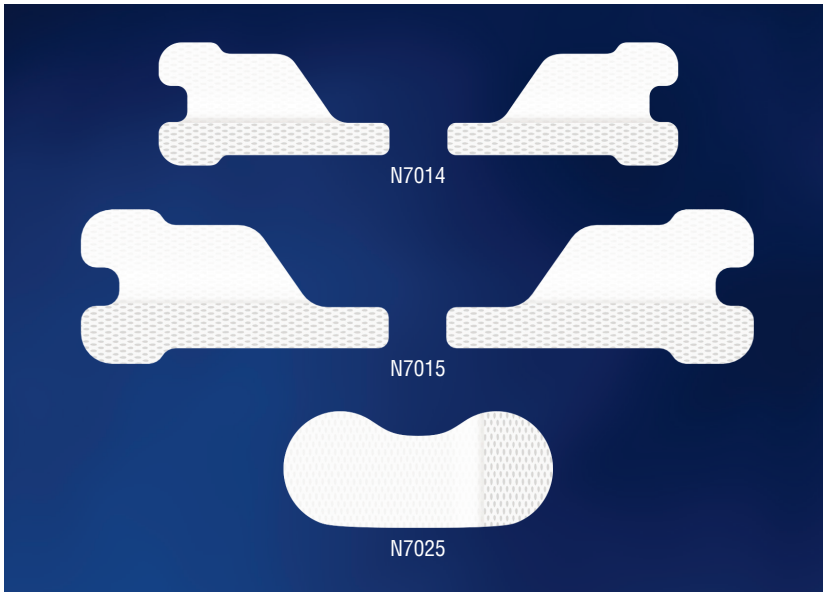

FEATURES & BENEFITS

- Skin friendly NeoBond[®] Hydrocolloid adhesive base
- Unique shape of original NeoSnug ideal for securing tubes close to the nares (N7014, N7015)
- NeoSnug Oral ideal for securing tubes under the lips (N7025)
- Fold over flap engineered to relieve stress on tubes
- Permanent hold flap keeps tubes securely in place
- Non-woven fabric contours to face or body
- Two per set with left and right tube holders

CAT. #	SIZE	QTY/UNIT
N7014-15	Micro	30/box*
N7015-15	Small	30/box*
N7025-40	Small Oral	40/box

*15 sets of 2 (left/right)

Patent Pending



NeoSnug[®]

TUBING HOLDER

INDICATIONS FOR USE

NeoSnug is intended to support, secure, and hold gastrointestinal tubes on patients' skin.

DIRECTIONS FOR USE

Preparation

Clean and dry skin where NeoSnug will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSnug will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

- 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
- 2 One at a time, peel NeoSnug from the clear, printed liner, place on skin, and hold in place for 60 seconds to ensure proper adhesion.
- 3 Fold down the flap and peel away the white, paper liner labeled "1" from the base of the NeoSnug. This will expose the adhesive surface.
- 4 Lay tubing across the base.
- 5 Peel away the white, paper liner labeled "2" from the flap and fold the flap over the tubing to secure.

Note: Once closed, the NeoSnug flap is engineered to remain permanently in place. You cannot open and close repeatedly.

Removal

- 1 Replace NeoSnug and attached tubing every 5 days or per hospital protocol, whichever is sooner.
- 2 With the tubing still secured, saturate NeoSnug with water or saline.
- 3 Slowly peel the NeoSnug away from the skin as you swab the skin with water or saline.
- 4 Remove tubing per hospital protocol and discard NeoSnug and tubing.

GENERAL INFORMATION

Product Materials

NeoBond Hydrocolloid, non-woven fabric, transfer tape (does not contact skin)
This product is not made with natural rubber latex or plasticizer DEHP.

Size

N7014 Micro closed: 38 mm x 20.4 mm
N7015 Small closed: 50 mm x 25 mm
N7025 Small Oral closed: 44.5 mm x 19 mm

Packaging

N7014, N7015: sets of two individually packaged
N7025: individually packaged
Package type: Polyethylene bag

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Choking hazard. NeoSnug should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Risk of ingestion or aspiration.

Discontinue immediately if skin irritation occurs.

NeoSnug will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Disposable. Do not reuse.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355

©2024 Neotech Products LLC. Product configuration is a trademark owned by and exclusively associated with Neotech.



Made in USA

M768 Rev B